micronized nasal douche  
douché nasale micronisée  
mikronisierte Nasendusche  
gemitroniseerde neusdouché

Cod. 822160 Rev. 11/2017

Air Liquide Medical Systems S.r.l.  
Via del Prati, 62 - 25073 Bovezzo (BS) ITALY  
Tel. (+39) 030.201.51.19 - Fax (+39) 030.2098329  
www.devices.alriquidemedicalhealthcare.com

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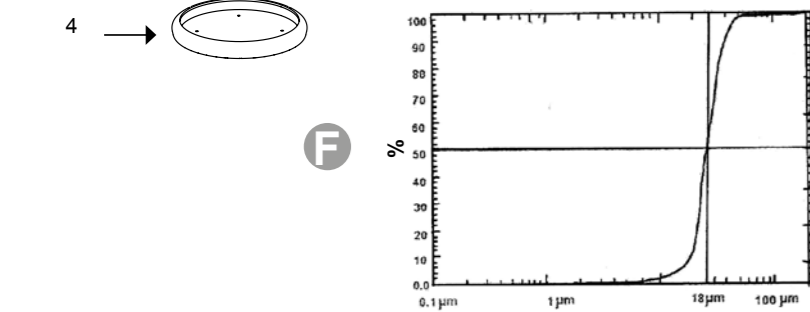
ENGLISH

#### 4 - CLEANING, DISINFECTING AND STERILISING

**⚠** Before proceeding, unplug the device and disconnect the Rinowash nasal douche and tube.

**⚠** If it is necessary to clean, disinfect or sterilise this medical device using a method other than that given in this section, contact the manufacturer first.

To prevent the risk of microbial contamination, the Rinowash atomized nasal douche must be cleaned and disinfected, and if necessary sterilised, after each treatment and before use, following the instructions carefully.

Before cleaning, disinfecting or sterilising any of the components, disassemble them as follows:

- Remove the tube.
- Remove the outer bell (7) by turning gently.
- Remove the atomiser (6) by pressing on the tab (Fig. E).
- Remove the base cap (4).

**⚠** Do not use a microwave oven for cleaning, disinfecting or sterilising.

**⚠** The seal at the base of the tank must not be removed.  
**⚠** The main section and the tank form a single component and cannot be disassembled.

#### 4.1 - CLEANING

The disassembled Rinowash can be washed in water at an approximate temperature of 40°C with the addition of detergent (refer to the manufacturer's instructions), then rinsed in running water. When all the parts have been cleaned, dry them with a soft cloth and place in the storage compartment.

#### 4.2 - DISINFECTING

After washing, all the components of the Rinowash nasal douche can be immersed in a cold disinfectant solution (e.g. a solution containing up to 2% sodium hypochlorite). Follow the manufacturer's instructions. The components of the nasal douche can be disinfected by boiling in a saucepan of water for a maximum of 10 minutes. Make sure the components are not in direct contact with the bottom of the saucepan.

**⚠** Boiling may alter the colour or appearance of the components, but this will not affect operation of the device.

#### 4.3 - STERILISING

All the components of the Rinowash nasal douche can be sterilised in an autoclave at 121°C for at least 15 minutes (but no more than 30 minutes), for up to 20 times.

#### 5 - MAINTENANCE

When cleaning, disinfecting or sterilising the nasal douche components, you can check them visually and determine when they need to be replaced. Repeated sterilisation may affect the materials of which the nasal douche is made, so the components should be inspected regularly. The nasal douche must be replaced when there are clear signs of deterioration.

#### 6 - PROBLEMS, POSSIBLE CAUSES AND SOLUTIONS

PROBLEMS	POSSIBLE CAUSES	SOLUTIONS
The Rinowash does not deliver atomised solution.	- The device is switched off. - There is no liquid in the tank. - The air tube is not connected securely or the Rinowash has not been assembled correctly. - The button has not been pressed.	- Switch on the device  - Pour liquid into the tank  - Connect the tube securely to the air intake on the device and the one on the Rinowash (3). Rotate the atomiser (6) slightly. Make sure all the parts have been assembled correctly. - Press the button.
Liquid leaks from the base of the bell.	- The outer bell (7) has not been fitted on properly and the seal does not hold.	- Fit on the outer bell (7) correctly, turning slightly. Make sure the whole seal is covered.

If after performing the above checks the Rinowash still does not work, have it checked by an authorized service centre.

#### 7 - TECHNICAL DATA

The Rinowash must not be used with ultra-sound or non-pneumatic aerosol therapy devices. The aerosol device must have the following characteristics.

Operating pressure	1-3.5 bar
Operating flow	5-20 l/min
MMAD(*) in this operating range	18 µm
Minimum Rinowash filling volume	1.5 ml
Maximum Rinowash filling volume	15 ml

N.B. The device uses an innovative technical solution for relieving pressure, which gives a constant delivery pressure (PATENTED). This means the MMAD(\*) value remains constant, irrespective of the supply pressure.

50% by volume of the atomised particles are less than the MMAD (Median Mass Aerodynamic Diameter). This value is indicative of the Rinowash's ability to produce particles suitable for a specific therapeutic treatment.

(\*)The MMAD value shown was measured using a MACH 2 API AEROSIZER and refers to the use of physiological solution (0.9% NaCl), so it may vary with the drug used. The value shown does not apply to suspended or high-viscosity drugs. Consult the drug supplier for further details.

The y-axis of the graphs (fig. F) represent the percentage of volume of atomised solution with sizes lower than the corresponding dimensional value expressed in µm on the x-axis.

#### 8 - DISPOSING OF THE DEVICE

The Rinowash atomised nasal douche is made of plastic materials and must be disposed of in accordance with the current regulations. Replaced components must be disposed of in a bin for plastic materials. The table below shows the kind of plastic of which each component is made. None of the materials used contains phthalates or traces of natural Latex.

COMPONENT	MATERIAL
Rinowash body (1)	Polycarbonate (some metal parts)
Atomiser (6)	Polycarbonate
Outer bell (7)	Polycarbonate and elastomer
Base cap (4)	Elastomer

#### 9 - OPERATING CONDITIONS

- Temperature range +5 °C to +40 °C
- Relative humidity 15% to 93%
- Atmospheric pressure 700 hPa to 1060 hPa

#### 10 - STORAGE CONDITIONS

- Temperature range -25 °C to +50 °C
- Relative humidity less than 93%
- Atmospheric pressure 700 hPa to 1060 hPa

#### 11 - SPARE PARTS

Only original spare parts must be used.

#### 12 - WARRANTY

The warranty only applies if accompanied by a receipt of purchase. The manufacturer is liable for the safety, reliability and performance of the device, provided that it is used in accordance with the instructions and for the intended purpose only.

The warranty only covers manufacturing defects. It does not cover damage due to normal wear and tear, incorrect use, tampering, or repairs not performed by the manufacturer or an authorised service centre. In all these cases, the manufacturer cannot be held liable and the warranty conditions are invalidated. If the defect is attributable to the manufacturing process, the device will be repaired or replaced at the manufacturer's discretion.

#### 13 - COPYRIGHT

The information contained in this booklet must not be used for any purpose other than the reason for which it is provided. This booklet is the property of Air Liquide Medical Systems S.r.l. and no part of it may be reproduced without the manufacturer's written permission. All rights reserved.

#### 14 - TECHNICAL UPDATES

Air Liquide Medical Systems S.r.l. periodically reviews and modifies all its medical devices in order to improve their performance, safety and reliability. The instruction booklets are updated accordingly to include any new or changed features. If the booklet accompanying the device gets damaged or mislaid, a replacement copy can be obtained from the manufacturer by quoting the data shown on the rating plate.

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