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Due to the inhalation and exhalation valve, the innovative design and the antistatic material, the device improves the therapy performed with MDI.

The device is composed of: a bottom (1) with universal opening for the MDI inhaler, holding chamber (2) inhalation and exhalation valve (3) upper body with mouthpiece (4) and air outlet (6) mask (5), if any.

The mask should be attached directly to the mouthpiece.

Spatial is available in the following versions:
- pediatric with mask for children between 0 and 6 years
- adult with mask: for patients over 6 years of age

Technical characteristics:
Chamber dimension: 160 ml
Dead space:
- pediatric with mask for children between 0 and 6 years: 48 ml
- adult with mask for patients over 6 years of age: 91 ml

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⚠ Read these instructions carefully before use and keep them on hand.

Intended use: The device is intended for use by patients under the supervision of a doctor. Spatial is a valved holding chamber suitable for use with metered-dose inhalers (MDI) and can be used by both children and adults, at home or in hospital. The device allows to improve the efficacy of drug administration at lung level by reducing side effects.

- The device cannot be sterilized in an autoclave.
- The repeated disinfection cycles may alter the colour or the appearance of the device. Always check the intact state of the device before use.
- It is advisable to replace the device after a year of continuous use.
- Read the instructions of the drug before use.
- There are no contraindications to the use of the device.
- Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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Symbols

Manufacturer	Carefully read the instructions for use	Read the warnings carefully	Temperature limits	Pressure limits	Relative humidity limits		
The presence of latex was not detected	The presence of phthalates was not detected	Dispose of the device properly	CE Marking	Production date	Lot number	Medical device	Product reference

Updates to technical features

In order to continuously improve performance, safety and reliability, all medical devices produced by Air Liquide Medical Systems S.r.l. are periodically reviewed and improved. Operating instruction

manuals are therefore amended to ensure constant compliance with the characteristics of the devices launched to market. If the instruction manual accompanying this device is lost, a copy corresponding to the supplied device may be

obtained from the manufacturer by specifying the data shown on the technical data label.

purposes other than originally intended. This manual is owned by Air Liquide Medical Systems S.r.l. and it may not be reproduced, either wholly or in part, without the prior written permission of Air Liquide Medical Systems S.r.l. All rights reserved.

Copyright

All information contained herein may not be used for

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
ENGLISH	INSTRUCTIONS FOR USE Before use, carefully examine the product making sure that it is assembled correctly and there are no foreign bodies or missing components. Remove any foreign objects prior to use.	Read the instructions supplied with the MDI. Remove the cover of the MDI. Insert the MDI into the bottom.	Place the mouthpiece between your lips or place the mask on your face so as to cover your nose and mouth and make sure the mask perfectly adheres to your face.	Press the MDI and inhale through your mouth, follow the breathing procedure indicated by your doctor, the exhaled air will come out from the opening (5) on the mouthpiece.
---------	---	---	---	---

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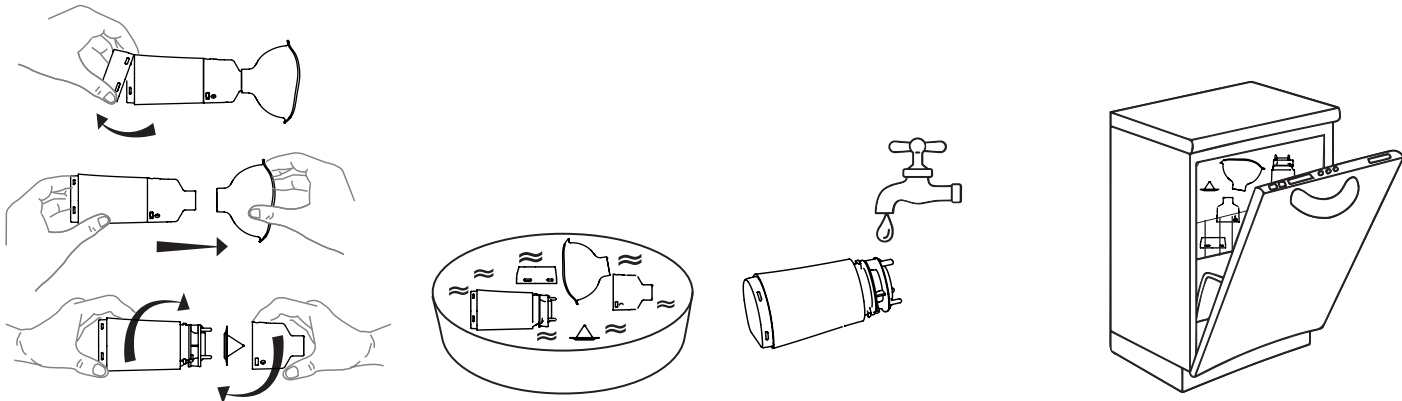
ENGLISH	CLEANING & DISINFECTION Remove the bottom. Detach the mask from the mouthpiece, turn the mouthpiece and remove the valve	Cleaning: Soak and wash all parts with warm water and neutral non-abrasive cleaner. Disinfection: after cleaning, soak the parts in a disinfectant solution (e.g. a solution containing sodium hypochlorite in concentrations not greater than 2%). Follow the instructions of the manufacturer of the disinfectant solution, and make sure it is compatible with the materials of the device. Rinse thoroughly with clean water.	The device, exception made for the mask, can be washed in the dishwasher with a special detergent at temperatures not greater than 70°C, placing it at the top of the dishwasher. The parts of the device, exception made for the mask, can disinfected boiling them in water for up to 10 minutes: the parts must not be in direct contact with the bottom of the pot. The parts of the device, exception made for the mask, can disinfected using a steam sterilizer.
---------	--	---	---

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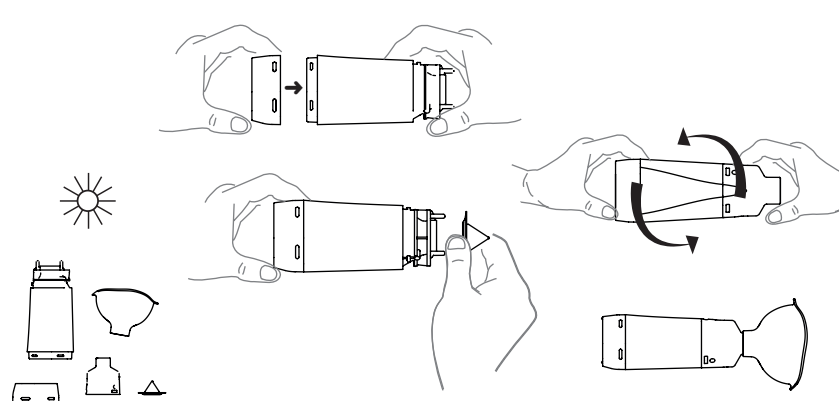ENGLISH	Allow to air dry. Do not use dry cloths to dry the parts of the device.	Make sure the pieces are dry before you reassemble the device. Hook the bottom to the device's body. Turn the valve on the chamber by threading the 3 pins into the corresponding holes.	Insert the mouthpiece onto the body and turn it to align the flat side. Assemble the mask on the mouthpiece, if any. The upper part of the mask (that corresponds to the nose) should be aligned with the wording.
---------	---	--	--

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Materials and disposal
The device and/or its parts should be disposed of in compliance with current regulations and environmental protection laws. Where there are no legal requirements, we recommend recycling. The table lists the materials of the various components of the device. All materials used are phthalate and Bisphenol-A free and the presence of natural rubber has not been detected.

PART	MATERIAL
Bottom	Anti-static polypropylene + rubber
Body	Anti-static polypropylene
Valve	Silicone
Mouthpiece	Polypropylene
Template	Non-toxic PVC

Conditions of use:
Temperature: from 5°C to 40°C
Relative humidity: from 15% to 93%
Atmospheric pressure: from 700 hPa to 1060 hPa
Storage conditions:
Temperature: from -25°C to 50°C
Relative humidity: from 30% to 95%
Atmospheric pressure: from 700 hPa to 1060 hPa

Warranty terms and conditions
The warranty is valid if it is accompanied by a receipt of purchase. The manufacturer is liable for the safety, the reliability and the performance of the device if the latter is used according to the instructions for use, its intended use and if all repairs are carried out at the manufacturer or at a laboratory authorized by the same. If the user fails to comply with the Operating Instructions and Warnings contained in this manual, the manufacturer's liability and warranty conditions shall be considered void. The warranty only covers defects caused by manufacturing, while it does not cover damages caused by normal wear and tear, any misuse or tampering. If the defect is attributable to manufacture, the device will be repaired or replaced by the manufacturer, as it deems necessary.

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