FULL FACE MASK

1 - APPLICATION

Respireo Hospital F Non Vented reusable is a non-vented full face mask. Respireo Hospital F Non Vented reusable is designed for use by patients with controlled non-invasive positive pressure ventilation (NIPPV). Respireo Hospital F Non Vented reusable is suitable for adult patients (>30Kg).

2 - IMPORTANT NOTES

- The mask must only be used with a ventilator equipped with an active exhalation valve or a respiratory circuit with an exhalation valve.
- The operating pressure range is 4-40 cmH₂O.
- If you experience discomfort or irritation, or suffer an allergic reaction to any of the components of the mask, please report this to your doctor or respiratory therapist.
- If oxygen is used with the ventilator, the supply must be turned off when the appliance is not in operation.
- Do not smoke or light naked flames when using oxygen.
- Do not use the mask in the presence of nausea or vomiting.
- The mask must be cleaned. For further details refer to section 7 - CLEANING THE MASK.
- Check the mask before use. If there is any visible damage, report it to the dealer.
- The mask must only be used in accordance with these instructions. No other instructions are applicable.
- Some of the components of the mask are small enough to be swallowed by children and must never be left unattended. An adult must always be present.

3 - COMPONENTS OF THE MASK (Fig. A)

a) mouth-and-nose cushion; b) main body; c) forehead support; d) forehead rest; e) adjusting knob; f) adjusting screw; g) cap; h) elbow; i) headgear; j) clip.

4 - HOW TO PUT ON THE MASK (Fig. B)

The mask comes fully assembled ready for use. Proceed as follows to put on the mask.
Turn the adjusting knob (e) until the forehead support (c) is fully extended (Fig. 1).
Hold the mask firmly in one hand and extend the headgear (i) with the other enough to pull it over your head (Fig. 2).
Place the mask gently on your face so that the cushion (a) fits snugly round the mouth and nose, and position the headgear (i) on your head (Fig. 3).
Clip the two clips (l) onto the main body (b) (Fig. 4).
Tighten the upper straps first, then the lower straps, until the mask fits on your face comfortably, without over tightening (Fig. 5).
Connect the mask to the ventilator circuit (Fig. 6).
After lying down, turn the adjusting knob (e) until the mask feels comfortable but still provides a good seal (Fig. 7).
Readjust the lower straps if necessary to eliminate any leaks, but make sure the mask is not too tight.

5 - HOW TO REMOVE THE MASK (Fig. C)

To remove the mask, press the button on the front (Fig. C) to release one of the clips (l). Lift the headgear over your head and remove the mask. The other straps do not need to be loosened or disconnected.

6 - HOW TO DISASSEMBLE THE MASK

Before disassembling the mask, disconnect the ventilator circuit.
Make a note of the position of the adjusting knob (e) for reference next time.
Unclip the two clips (l) from the main body (b).
Undo the four straps and detach the headgear (i) from the main body (b) and the clips (l).
Pull the elbow (h) off the main body (b).
Remove the cushion (a) from the main body (b). Remove the cap (g) from the main body (b).
Detach the forehead rest (d) from the forehead support (c).

7 - HOW TO CLEAN THE MASK

7.1 - DAILY CLEANING FOR REUSE BY THE SAME PATIENT
First wash your hands thoroughly.
Disassemble all the components (see HOW TO DISASSEMBLE THE MASK) and wash carefully in warm water at about 30°C, using mild soap. Rinse thoroughly under the tap and leave in the air to dry, but not in direct sunlight.





7.2 - REPROCESSING THE MASK FOR RE-USE WITH OTHER PATIENTS

N.B. The mask must always be regenerated before being used by another patient.
The mask can be regenerated up to 20 times, using one of the following methods:
• high-level thermal disinfection;
• high-level chemical disinfection;
• sterilisation in an autoclave at 121°C for no more than 20 minutes.

Full details of the mask regeneration process can be obtained by logging on to www.device.airliquidehealthcare.com or by contacting your dealer.

8 - PRECAUTIONS

Do not clean the mask with solutions containing alcohol, aromatic compounds, humidifiers, antibacterial agents or aromatic oils. Such solutions would damage the product and reduce its life. Prolonged exposure to direct sunlight or heat may cause damage to the mask.
If any visible damage is noted (cracks, tears, etc.), the damaged part must be replaced.
Check carefully to make sure there are no soap deposits or foreign bodies inside the mask.

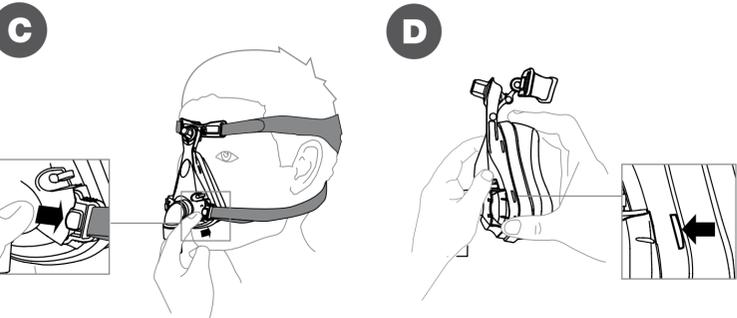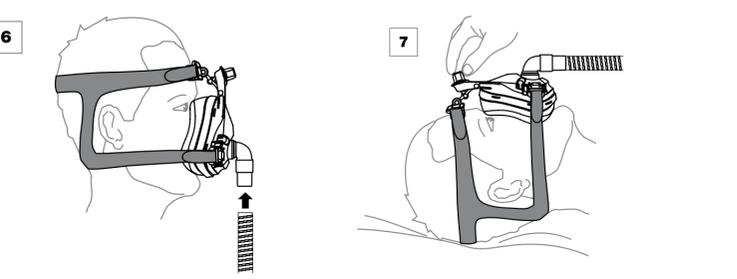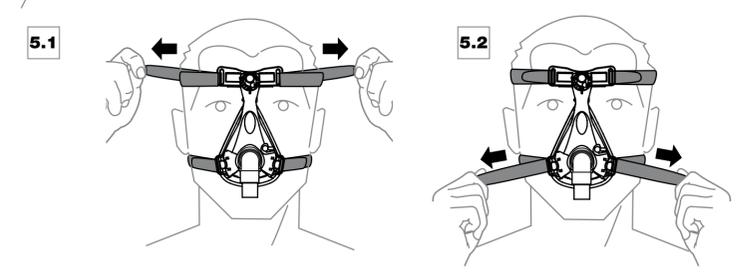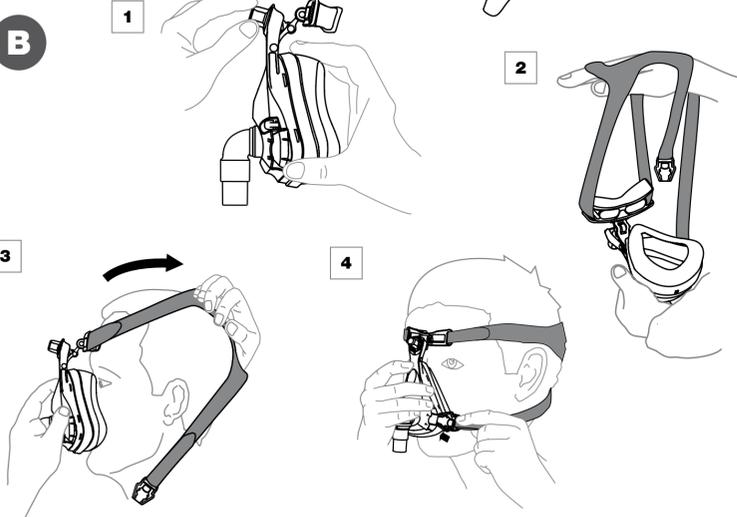9 - HOW TO REASSEMBLE THE MASK

Insert the elbow (h) in the main body (b) of the mask, pressing gently. Remove the forehead rest (d) on the forehead support (c), keeping the curved edge facing upwards.
Fit the grooved side of the cushion (a) into the frame of the main body (b). Fit on the top first, then the sides of the mask, pressing the cushion in until the whole edge is up against the six stops on the main body (Fig. D). Fit on the cap (g).
Pass the two lower straps of the headgear (i) through the slots in the clips (l). Attach the two upper straps of the headgear (i) to the main body (b). Clip the two clips (l) back onto the main body (b).
Turn the adjusting knob (e) to the reference position you noted previously.

10 - PRESSURE/OXYGEN PORT

If additional oxygen or a pressure reading is required, use the port in the main body (b).
Open the cap (g) by lifting upwards, without removing it.
Push the pressure/oxygen tube firmly into the port.

11 - TROUBLE-SHOOTING CHART

PROBLEMS	POSSIBLE CAUSES	SOLUTIONS
• No air or insufficient air comes out of the mask	• Flow generator off • Flow generator regulated wrongly • Patient circuit tube not inserted	• Switch on the flow generator • Regulate the flow generator • Connect the tube to the flow generator and mask
• The mask is noisy	• Cushion mounted incorrectly and air leak	• Mount the cushion correctly
• The mask does not fit the face snugly	• Mask applied incorrectly	• Reposition the headgear

12 - DISPOSAL INSTRUCTIONS

The components of the mask are made of the following materials and must be disposed of in accordance with the applicable laws.

COMPONENT	MATERIAL
Main body	Polycarbonate
Forehead support	Polycarbonate
Adjusting knob	Nylon
Adjusting screw	Polycarbonate
Elbow	Polycarbonate
Forehead rest	Silicone
Cushion	Silicone
Cap	Silicone
Headgear	Nylon/polyurethane
Clip	Nylon

13 - TECHNICAL DATA

Normative references	CE marking in accordance with European Directive 93/42/EEC, class Ila and subsequent amendments EN ISO 14971, ISO 5356-1, ISO 10993-1, EN ISO 17510-2, EN ISO 17664
Operating pressure	4-40 cmH ₂ O
Temperature range	5°C - 40°C
Dead space volume inside the mask	215 ml (Large)

14 - WARRANTY

This product by Air Liquide Medical Systems S.r.l. is guaranteed against material and manufacturing defects for a period of 120 days as from the date of purchase, provided it is used in accordance with the operating instructions and for the intended purpose only.
If any malfunctions occur during the warranty period under normal use conditions, Air Liquide Medical Systems S.r.l. will, at its own discretion, repair or replace the defective product or any of its components.
The manufacturer is liable for the safety, reliability and performance of the apparatus, provided that it is used in accordance with the instructions and for the intended purpose only, and any repairs are carried out by the manufacturer or an approved service centre.
The original packing must be kept for any return throughout the warranty period.
Warranty claims based on product defects must be reported to your dealer.
For further information on the warranty terms and conditions, please contact your dealer.

COPYRIGHT

The information contained in this booklet must not be used for any purpose other than the reason for which it is provided.
This booklet is the property of Air Liquide Medical Systems S.r.l. and no part of it may be reproduced without the manufacturer's written permission. All rights reserved.

TECHNICAL UPDATES

Air Liquide Medical Systems S.r.l. periodically reviews and modifies all its medical devices in order to improve their performance, safety and reliability. The instruction booklets are updated accordingly to include any new or changed features.
If the booklet accompanying the apparatus gets damaged or mislaid, a replacement copy can be obtained from the manufacturer by quoting the data shown on the rating plate.

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